

# Summary of clinical trials of CarciReagent tester kit in China

## (Experiment Test – full version)

(This test is done according to EN13612:2002)  
File No.: SCT/ZYB/2021

### Introduction

In this test, we want to prove if this diagnostic medical device can help with his result of indicative volume of tyrosine in human urine for detection of possible malignant tumor diseases. We used total of 8.078 urine samples (divided into 2 groups - Malignant tumor group (patients diagnosed with cancer by the hospital, mainly including: gastrointestinal malignant tumor, liver cancer, nasopharyngeal carcinoma, malignant lymphoma, breast cancer, gynecological malignant tumor, lung cancer, etc.); and Non-malignant tumor group (confirmed non-cancer patients, including: patients with tyrosine-related diseases, patients with common diseases, and normal health people). Our purpose was to set relation between level of monohydroxyphenol metabolites (tyrosine) and malignant/nonmalignant diseases.

### Content:

- 1, Name of in vitro diagnostic device for self-testing
- 2, Description of in vitro diagnostic device for self-testing
- 3, Intended use of in vitro diagnostic device for self-testing
- 4, Principle and function of in vitro diagnostic device for self-testing
- 5, Objective of the functional evaluation of in vitro diagnostic device for self-testing
- 6, Plan of functional evaluation of in vitro diagnostic device for self-testing
- 6a, Responsible person of functional evaluation of in vitro diagnostic device for self-testing
- 6b, List of laboratories used for functional evaluation of in vitro diagnostic device for self-testing
- 6c, Timetable for functional evaluation of in vitro diagnostic device for self-testing
- 6d, Number of samples taken, and tests performed for functional evaluation of in vitro diagnostic device for self-testing
- 6e, Instructions for use, including a description of the conditions of use for functional evaluation of in vitro diagnostic device for self-testing
- 6f, Data on the function of the in vitro diagnostic device for self-testing
- 6g, Documentation of functional evaluation of in vitro diagnostic device for self-testing
- 7, Basic information on the intention to functional evaluation of in vitro diagnostic device for self-testing
- 8, Design of experimental setup of functional evaluation of in vitro diagnostic device for self-testing
- 9, Structure of functional evaluation of in vitro diagnostic device for self-testing
- 10, Records of tests of functional evaluation of in vitro diagnostic device for self-testing
- 11, Observations and unexpected results by functional evaluation of in vitro diagnostic device for self-testing
- 12, Evaluation of individual experimental tests and functional evaluation report of in vitro diagnostic device for self-testing
- 13, Changes during the functional evaluation study or re-evaluation
- 14, Protection and safety of the persons under investigation
- 15, References and list of documents

---

ID: 082 90 458

Bank CZK: FIO Bank; IBAN: CZ24 2010 0000 0023 0174 9129; BIC: FIOBCZPPXXX

Bank USD: FIO Bank; IBAN: CZ49 2010 0000 0021 0174 9132; BIC: FIOBCZPPXXX

Bank EUR: FIO Bank; IBAN: CZ39 2010 0000 0028 0174 9133; BIC: FIOBCZPPXXX

[www.carcireagent.com](http://www.carcireagent.com)

[info@carcireagent.com](mailto:info@carcireagent.com)

### **1, Name of in vitro diagnostic device for self-testing**

CarciReagent

### **2, Description of in vitro diagnostic device for self-testing**

Test kit for the detection of the approximate amount of monohydroxyphenol metabolites (tyrosine) in urine (Lay semi-quantitative test for self-testing) (Chemical chromogenic method)

### **3, Intended use of the in vitro diagnostic device for self-testing**

"CarciReagent - an in vitro diagnostic device for self-testing, intended for the detection of monohydroxyphenol metabolites (tyrosine) and its approximate amount in the patient's urine (Lay semiquantitative test)"

### **4, Principle and function of in vitro diagnostic device for self-testing**

The basic principle of the test is based on the improved method of Millon's reagent (Millon's reagent), which monitors the increased amount of tyrosine (monohydric phenolic amino acids and their metabolites) in the urine. By changing the color of the mixture in the ampoule after adding 3 ml of morning urine (middle urine stream), the reaction color cascade can be used to determine if urine samples contain increased amounts of these metabolites. The reagents in the ampoule and the tyrosine content in the urine show a characteristic chromogenic response that can be used for the clinical diagnosis of intracellular metabolic abnormalities (detection of possible changes or disorders in metabolism within the human cell). The determined approximate content of tyrosine in urine (according to the attached table from 0–2000 mg per liter of urine) reacts with the chemical reagent and, depending on its amount, turns colored. According to the enclosed color scale, the test result can be read from No. 1 to No. 8. The result from No. 1 to No. 3 is the amount of tyrosine in the normal concentration, so the result is considered negative, and no increased amount of tyrosine has been demonstrated. With results No. 4 and No. 5, it is already a positive finding of an increased amount of tyrosine in the urine. If the tyrosine concentration is higher than 500 mg per liter of urine, ie. result No. 6, No. 7, No. 8, this is a positive result, a high content of tyrosine in the urine may indicate a more serious disease. In case of positive results, it is recommended to perform a more thorough examination by a general practitioner in order to exclude the risk of a possible serious illness.

### **5, Objective of the functional evaluation of in vitro diagnostic device for self-testing**

In this test, we want to proof if CarciReagent tester can help with his result of indicative volume of tyrosine in human urine for detection of possible malignant tumor diseases. Looking for particular kind of cancer was not objective of this test. Our purpose was to set relation between level of monohydroxyphenol metabolites (tyrosine) and malignant/nonmalignant diseases.

---

ID: 082 90 458

Bank CZK: FIO Bank; IBAN: CZ24 2010 0000 0023 0174 9129; BIC: FIOBCZPPXXX

Bank USD: FIO Bank; IBAN: CZ49 2010 0000 0021 0174 9132; BIC: FIOBCZPPXXX

Bank EUR: FIO Bank; IBAN: CZ39 2010 0000 0028 0174 9133; BIC: FIOBCZPPXXX

[www.carcireagent.com](http://www.carcireagent.com)

[info@carcireagent.com](mailto:info@carcireagent.com)

## **6, Plan of functional evaluation of in vitro diagnostic device for self-testing**

### **6a, Responsible person of functional evaluation of in vitro diagnostic device for self-testing**

Mr. Hu Min (Quality Inspector) is appointed as the person responsible for conducting the in vitro diagnostic device function evaluation.

The responsible persons who performed the laboratory samples, performed the tests, and evaluated the results are fully qualified to work in the laboratory. Laboratories are responsible for performing the required experimental tests and have the necessary certification. The laboratories used to perform the tests are part of large hospitals, which are always listed in the individual test reports.

### **6b, List of laboratories used for functional evaluation of in vitro diagnostic device for self-testing**

Second Affiliated Hospital of Nanchang University, Affiliated Hospital of Jiangxi University of Traditional Chinese Medicine, Second Affiliated Hospital of Nanjing Medical University, Shanghai Tenth People's Hospital, Shanghai Changhai Hospital

### **6c, Timetable for functional evaluation of in vitro diagnostic device for self-testing**

1.6.-30.9.2021

### **6d, Number of samples taken, and tests performed for functional evaluation of in vitro diagnostic device for self-testing**

8.078 samples of urine (Age of people: 18-60)

For testing we used these batches LOT 20190303, LOT 20190501, LOT 20190702 (all valid for three years). Because three batches were used, we made three control groups, there were Control group 1 - 2700 tests from batch 20190303, Control group 2 - 2700 tests from batch 20190501, and Control group 3 - 2700 tests from batch 20190702. From Group 3 were 22 tests from batch 20190702 not used in this experiment.

### **6e, Instructions for use, including a description of the conditions of use for functional evaluation of in vitro diagnostic device for self-testing**

As part of the tests used in this evaluation of the function of the in vitro diagnostic device for self-testing, the laboratories were provided with standard instructions for use (Chinese version). In these tests, only urine collection was performed by the patients themselves (lay people), then the collected urine was immediately taken over by the laboratory to perform the test, the test itself and its evaluation was already performed by professional laboratory staff.

### **6f, Data on the function of the in vitro diagnostic device for self-testing**

All function data are described in the chapters above and in particular in chapter "5, Objective of the evaluation of the function of the in vitro diagnostic device for self-testing".

## **6g, Documentation of functional evaluation of in vitro diagnostic device for self-testing**

All documentation used is part of the technical documentation of the in vitro diagnostic device CarciReagent and is in the possession of the manufacturer (CNEU MEDICAL s.r.o.).

## **7, Basic information on the intention to functional evaluation of in vitro diagnostic device for self-testing**

The manufacturer provided full information on the purpose of the in vitro diagnostic device, its detection principle, its proposed use, the method of evaluation of the test result and the interpretation of the result. Information was provided to those who collected the urine, performed the test, and evaluated the result and fully understood the function and use of the CarciReagent in vitro diagnostic device.

## **8, Design of experimental setup of functional evaluation of in vitro diagnostic device for self-testing**

The procedure for validation tests is as follows:

- 1, Selection of tested persons according to the intention of individual evaluations (with regard to health status, age, location of testing)
- 2, Urine samples collection: A total of 8,078 fresh and clean urine samples were collected and divided into groups according to different health status.
- 3, Sample test: Suck 3ml of the urine sample with a urine pipette, add it into the ampoule bottle containing the reagent, and let it stand.
- 4, After the reaction time is reached, observe the color of the precipitate and compare it with the standard color chart, and finally record the accurate color result. The judgment criteria are:
  - a, Negative: The reaction result is colorless/pale yellow/reddish.
  - b, Positive: The reaction result is light red/pink/rose red/brown red/dark rust red.
- 5, Carrying out the evaluation of all performed tests from all five location with regard to the goal of the given function evaluation.
- 6, Processing and passing on the conclusions of the evaluation of the function, including informing about unexpected results or other undesirable situations during the implementation of the experiment.

Aspects of tests for validation of in vitro diagnostic device function evaluation for self-testing:

Aspects that could affect the results of the CarciReagent in vitro diagnostic device performance evaluation were taken into account during the individual experimental tests. Emphasis was placed on qualified selection of test subjects, proper urine collection, adherence to the test procedure and its evaluation. The stability of the test set is set at 3 years. The persons who performed the testing and evaluation were acquainted with the aim of the individual experimental tests.

## **9, Structure of functional evaluation of in vitro diagnostic device for self-testing**

A total of 8,078 urine samples were tested, including:

Tested group 1: Malignant tumor group (patients diagnosed with cancer by the hospital, mainly including: gastrointestinal malignant tumor, liver cancer, nasopharyngeal carcinoma, malignant lymphoma, breast cancer, gynecological malignant tumor, lung cancer, etc.), a total of 4375 samples



were tested among them, 4230 cases were positive, and 145 cases were negative. The average positive rate was 96.69%.

Tested group 2: The non-malignant tumor group (confirmed non-cancer patients, including: patients with tyrosine-related diseases, patients with common diseases, and normal health people) tested a total of 3703 samples, of which 124 were positive and 3579 were negative, with an average positive rate is 3.35%. If we consider only normal health people in total 2662 cases, and 16 positive cases, we reach the average positive rate only 0.6%.

Tables with results:

<b>Control group 1:</b>			<b>Batch 20190303</b>		
<b>Disease</b>		<b>Case</b>	<b>Result</b>		<b>Positive rate</b>
			<b>Positive</b>	<b>Negative</b>	
<b>Tested group 1</b>	<b>Cancer patient group</b>	1459	1410	49	96,64%
<b>Tested group 2</b>	<b>Pigmentary disorders(chloasma)</b>	24	5	19	20,83%
	<b>Diabetes</b>	32	8	24	25,00%
	<b>Gastritis</b>	56	9	47	16,07%
	<b>Gastric ulcer</b>	27	4	23	14,81%
	<b>Parkinson's disease</b>	14	0	14	0,00%
	<b>Depression</b>	13	0	13	0,00%
	<b>Albinism</b>	8	0	8	0,00%
	<b>Black aciduria and Phenylketonuria</b>	4	0	4	0,00%
	<b>Esophagitis</b>	26	2	24	7,69%
	<b>Enteritis</b>	23	2	21	8,70%
	<b>Benign prostatic hyperplasia</b>	10	1	9	10,00%
	<b>Tuberculosis</b>	20	3	17	15,00%
	<b>Cholecystitis</b>	39	4	35	10,26%
	<b>Viral hepatitis</b>	35	4	31	11,43%
	<b>Lung infection</b>	30	3	27	10,00%
	<b>Normal health population</b>	880	5	875	0,57%



Control group 2:				Batch 20190501	
Tested group 1	Disease	Case	Result		Positive rate
			Positive	Negative	
	<b>Cancer patient group</b>	1458	1409	49	96,64%
Tested group 2	Pigmentary disorders(chloasma)	22	5	17	22,73%
	Diabetes	30	6	24	20,00%
	Gastritis	54	7	47	12,96%
	Gastric ulcer	25	3	22	12,00%
	Parkinson's disease	12	0	12	0,00%
	Depression	11	0	11	0,00%
	Albinism	6	0	6	0,00%
	Black aciduria and Phenylketonuria	3	0	3	0,00%
	Esophagitis	24	1	23	4,17%
	Enteritis	21	0	21	0,00%
	Benign prostatic hyperplasia	8	0	8	0,00%
	Tuberculosis	18	1	17	5,56%
	Cholecystitis	37	2	35	5,41%
	Viral hepatitis	33	2	31	6,06%
	Lung infection	28	1	27	3,57%
	<b>Normal health population</b>	910	4	906	0,44%

Control group 3:				Batch 20190702	
Tested group 1	Disease	Case	Result		Positive rate
			Positive	Negative	
	<b>Cancer patient group</b>	1458	1411	47	96,78%
Tested group 2	Pigmentary disorders(chloasma)	22	5	17	22,73%
	Diabetes	31	6	25	19,35%
	Gastritis	56	9	47	16,07%
	Gastric ulcer	26	4	22	15,38%
	Parkinson's disease	14	0	14	0,00%
	Depression	12	0	12	0,00%
	Albinism	8	0	8	0,00%
	Black aciduria and Phenylketonuria	3	0	3	0,00%
	Esophagitis	24	1	23	4,17%
	Enteritis	22	1	21	4,55%
	Benign prostatic hyperplasia	10	1	9	10,00%
	Tuberculosis	18	1	17	5,56%
	Cholecystitis	39	2	37	5,13%
	Viral hepatitis	34	3	31	8,82%
	Lung infection	29	2	27	6,90%
	<b>Normal health population</b>	872	7	865	0,80%

ID: 082 90 458

Bank CZK: FIO Bank; IBAN: CZ24 2010 0000 0023 0174 9129; BIC: FIOBCZPPXXX

Bank USD: FIO Bank; IBAN: CZ49 2010 0000 0021 0174 9132; BIC: FIOBCZPPXXX

Bank EUR: FIO Bank; IBAN: CZ39 2010 0000 0028 0174 9133; BIC: FIOBCZPPXXX

[www.carcireagent.com](http://www.carcireagent.com)[info@carcireagent.com](mailto:info@carcireagent.com)

<b>Summary of results (3 control groups)</b>					
	Disease	Case	Result		Positive rate
			Positive	Negative	
<b>Tested group 1</b>	<b>Cancer patient group</b>	<b>4375</b>	<b>4230</b>	<b>145</b>	<b>96,69%</b>
<b>Tested group 2</b>	<b>Pigmentary disorders(chloasma)</b>	68	15	53	22,06%
	<b>Diabetes</b>	93	20	73	21,51%
	<b>Gastritis</b>	166	25	141	15,06%
	<b>Gastric ulcer</b>	78	11	67	14,10%
	<b>Parkinson's disease</b>	40	0	40	0,00%
	<b>Depression</b>	36	0	36	0,00%
	<b>Albinism</b>	22	0	22	0,00%
	<b>Black aciduria and Phenylketonuria</b>	10	0	10	0,00%
	<b>Esophagitis</b>	74	4	70	5,41%
	<b>Enteritis</b>	66	3	63	4,55%
	<b>Benign prostatic hyperplasia</b>	28	2	26	7,14%
	<b>Tuberculosis</b>	56	5	51	8,93%
	<b>Cholecystitis</b>	115	8	107	6,96%
	<b>Viral hepatitis</b>	102	9	93	8,82%
	<b>Lung infection</b>	87	6	81	6,90%
	<b>Normal health population</b>	2662	16	2646	0,60%

#### 10, Records of tests of functional evaluation of in vitro diagnostic device for self-testing

All detailed documentation of the records of functional tests performed in the laboratories of various hospitals is available only to these institutions. The procedures were performed in accordance with applicable laws and regulations. After performing the tests, the laboratories prepared summaries and evaluations with data that are not subject to personal data protection. These have been received by the manufacturer and are part of the technical documentation for the in vitro diagnostic device.

#### 11, Observations and unexpected results by functional evaluation of in vitro diagnostic device for self-testing

Unexpected results were not observed during the experimental tests, no deficiencies were reported with the test kits, and the tests did not need to be repeated.

#### 12, Evaluation of individual experimental tests and functional evaluation report of in vitro diagnostic device for self-testing

After testing, the malignant tumor group (patients diagnosed with cancer by the hospital) had an average positive rate of 96.69%, and the non-malignant tumor group (diagnosed non-cancer patients, including: patients with tyrosine-related diseases, patients with common diseases, and normal health people) had an average positive rate of 3.35%. And by only normal health people was average positive rate by 0,6%. The positive rate of the malignant tumor group was significantly higher

than that of the non-malignant tumor group. The difference between the two detection results was statistically significant.

Malignant tumors are the most important cause of positive test results, and they are statistically highly probable.

In the non-malignant tumor group, the positive rate of patients with pigment disorders (chloasma), gastritis, gastric ulcer and diabetes is relatively high, which is one of the causes that may lead to a positive test result. However, the overall level of the positive proportion is low, which is a statistically small probability situation.

In the non-malignant tumor group, the positive rate of other types of common diseases and people in normal health status is very low. The analysis may be mainly due to the false positive results caused by the stress response and interference factors of the human immune system, and there is no statistics significance.

After a large number of clinical tests of this reagent, the positive detection rate of malignant tumors is significant. According to the statistical analysis, the positive results have a great probability to detect malignant tumor diseases. The positive results of other diseases are very low, or none.

### **13, Changes during the functional evaluation study or re-evaluation**

No changes were made during the evaluation study or re-evaluation.

### **14, Protection and safety of the persons under investigation**

The protection and safety of the persons under investigation were fully ensured and no person was exposed to any danger. The protection of the personal data of the persons who took part in the testing was fully ensured.

### **15, References and list of documents**

Peng Xiangmin, Jiang Yuping. Tyrosine hydroxylase and Parkinson's disease. Chinese Clinical Neuroscience, 2002, 10(1): 105-108.

<https://d.wanfangdata.com.cn/periodical/zglcsjcx200201041>

Wu Hong, Kaida Jiang, Jianying Qiu et al. Association analysis of tyrosine kinase receptor B gene polymorphism and depression. Journal of Shanghai Jiaotong University (Medical Edition), 2010, 30(6): 620-623.

<https://d.wanfangdata.com.cn/periodical/ChlQZXJpb2RyY2FsQ0hJTmV3UzlwMjEwNjE2EhNzaGRleWtkeHhiMjAxMDA2MDA0Gghkd25qZm01eA%3D%3D>

Peng Chen, Deng Weiping. Research progress in albinism. International Journal of Dermatology and Venereology. 2014, 40(1): 26-28.

<https://d.wanfangdata.com.cn/periodical/ChlQZXJpb2RyY2FsQ0hJTmV3UzlwMjEwNjE2EhNnd3l4LXBmeGJ4MjAxNDAxMDEwGghncHppMTllcw%3D%3D>

Chen Qingxi, Song Kangkang. Research Progress in Tyrosinase. Journal of Xiamen University (Natural Science Edition) 2006, 45(5): 731-737.

<https://d.wanfangdata.com.cn/periodical/ChlQZXJpb2RyY2FsQ0hJTmV3UzlwMjEwNjE2Eg94bWR4eGlyMDA2MDUwMzMaCGxaWF2aGpm>

---

ID: 082 90 458

Bank CZK: FIO Bank; IBAN: CZ24 2010 0000 0023 0174 9129; BIC: FIOBCZPPXXX

Bank USD: FIO Bank; IBAN: CZ49 2010 0000 0021 0174 9132; BIC: FIOBCZPPXXX

Bank EUR: FIO Bank; IBAN: CZ39 2010 0000 0028 0174 9133; BIC: FIOBCZPPXXX

[www.carcireagent.com](http://www.carcireagent.com)

[info@carcireagent.com](mailto:info@carcireagent.com)



Zhang Zhi, He Yunshao. Progress in molecular genetics of phenylketonuria. *Heredity*, 2004, 26 (5): 729-734.

<https://d.wanfangdata.com.cn/periodical/ChIQZXJpb2RpY2FsQ0hJTmV3UzlwMjEwNjE2Egt5YzlwMDQwNTAzMR0lb2QxNHZqN2s%3D>

Han Lianshu, Ye Jun, Qiu Wenjuan, etc. The application of succinylacetone detection in blood and urine in the diagnosis of tyrosinemia type I. *Chinese Journal of Pediatrics*, 2012, 50(2): 126-130.

<https://d.wanfangdata.com.cn/periodical/ChIQZXJpb2RpY2FsQ0hJTmV3UzlwMjEwNjE2Eg16aGVrMjAxMjAyMDEwGghsMmx5aXBhcA%3D%3D>

Luo Yang, Wang Jue, Zhang Xue, etc. The application of the detection of amino acid metabolites in urine in the screening of malignant tumors. *Journal of Modern Laboratory Medicine*, 2009, 24(2): 66-69.

<https://d.wanfangdata.com.cn/periodical/ChIQZXJpb2RpY2FsQ0hJTmV3UzlwMjEwNjE2Eg9zeHl4ankyMDA5MDIwMTcaCDI0b3hpcTJr>

Huang Xuemei, Wu Lixiang, Lu Zilan, etc. The application value of urine p-hydroxyphenylalanine in the early prediction of malignant tumors. *Laboratory Medicine and Clinics*, 2015, 12(16): 2333-2335.

<https://d.wanfangdata.com.cn/periodical/ChIQZXJpb2RpY2FsQ0hJTmV3UzlwMjEwNjE2EhBqeXl4eWxjMjAxNTE2MDE0GghpdjR1ZXQ3Ng%3D%3D>

Zhang Huijuan, Liu Xiaomin, Du Baiyan, etc. The relationship between nitrotyrosine level and age, body mass index and other related factors in diabetic patients. *China Clinical Rehabilitation*, 2004, 10(36): 92-94.

<https://www.xueshu.com/zgzzgcyj/200636/26655270.html>